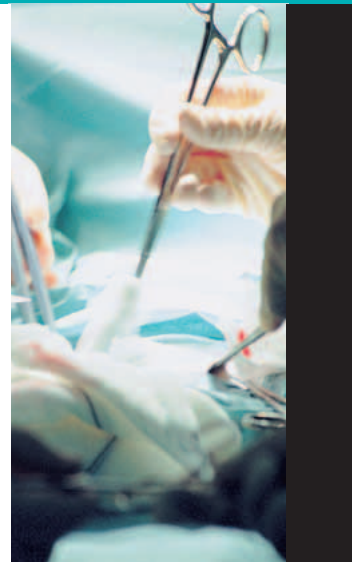
.steute

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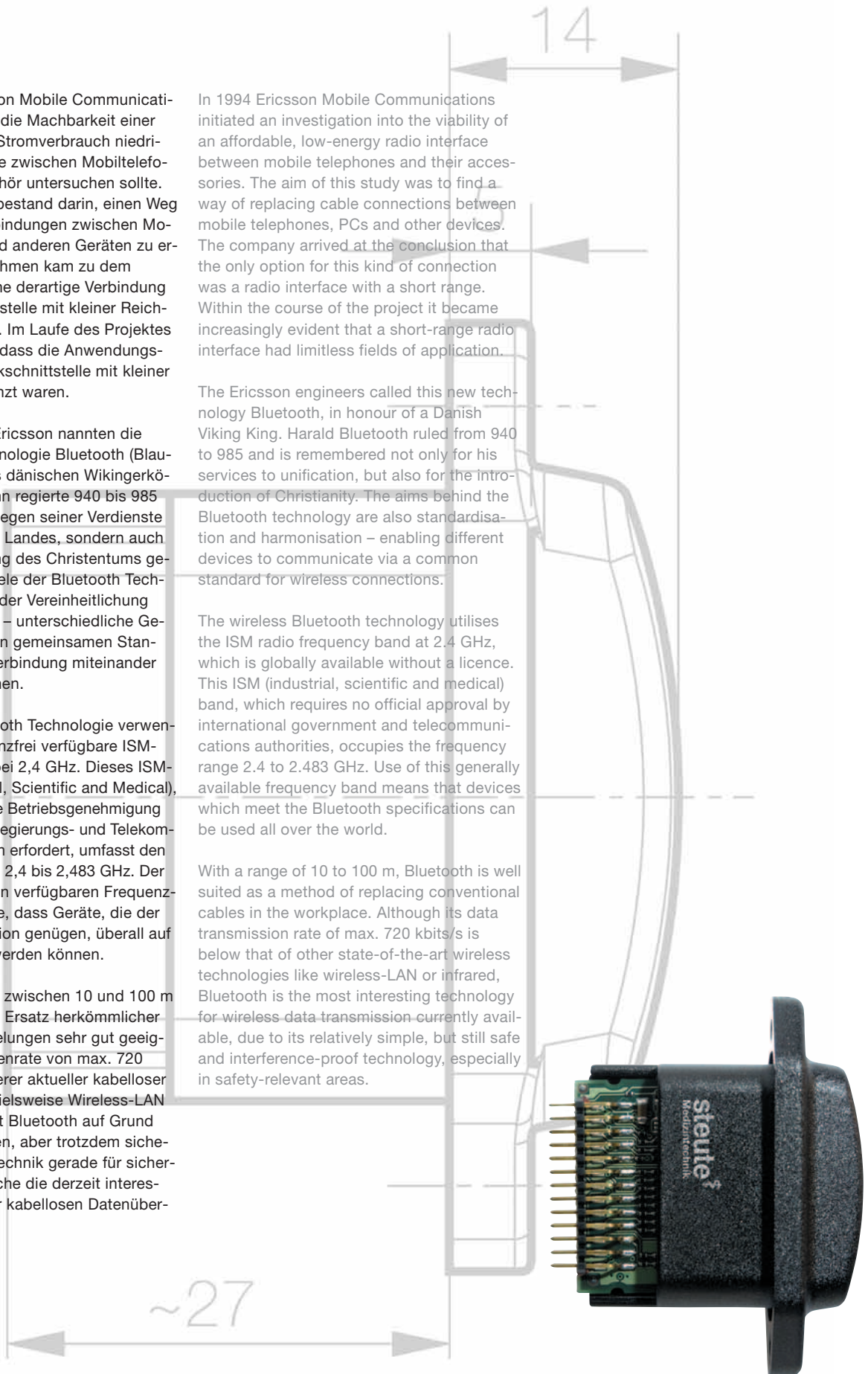
In 1994 Ericsson Mobile Communications initiated an investigation into the viability of an affordable, low-energy radio interface between mobile telephones and their accessories. The aim of this study was to find a way of replacing cable connections between mobile telephones, PCs and other devices. The company arrived at the conclusion that the only option for this kind of connection was a radio interface with a short range. Within the course of the project it became increasingly evident that a short-range radio interface had limitless fields of application.

The Ericsson engineers called this new technology Bluetooth, in honour of a Danish Viking King. Harald Bluetooth ruled from 940 to 985 and is remembered not only for his services to unification, but also for the introduction of Christianity. The aims behind the Bluetooth technology are also standardisation and harmonisation – enabling different devices to communicate via a common standard for wireless connections.

The wireless Bluetooth technology utilises the ISM radio frequency band at 2.4 GHz, which is globally available without a licence. This ISM (industrial, scientific and medical) band, which requires no official approval by international government and telecommunications authorities, occupies the frequency range 2.4 to 2.483 GHz. Use of this generally available frequency band means that devices which meet the Bluetooth specifications can be used all over the world.

With a range of 10 to 100 m, Bluetooth is well suited as a method of replacing conventional cables in the workplace. Although its data transmission rate of max. 720 kbits/s is below that of other state-of-the-art wireless technologies like wireless-LAN or infrared, Bluetooth is the most interesting technology for wireless data transmission currently available, due to its relatively simple, but still safe and interference-proof technology, especially in safety-relevant areas.

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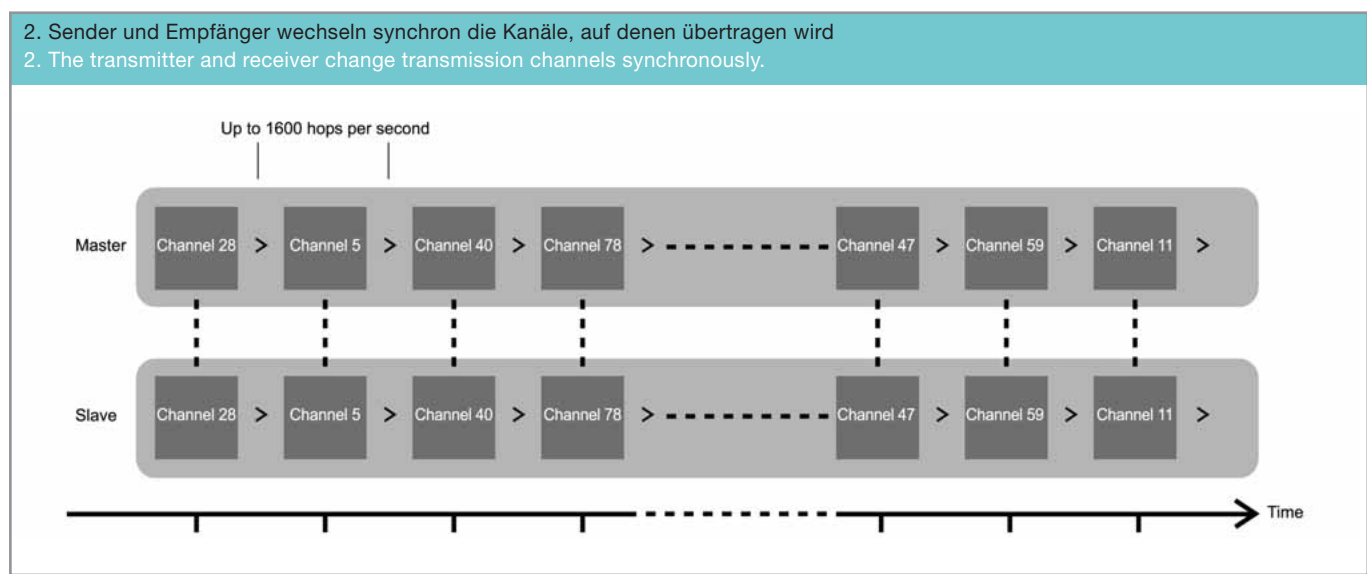
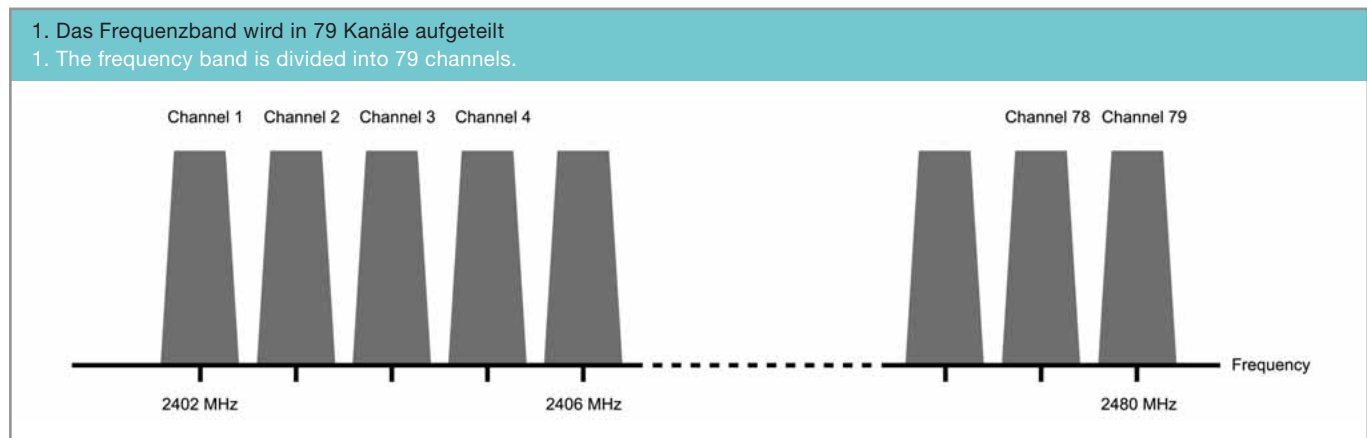
Why wireless?

With our new wireless foot controls, we at **steute** are aiming to increase safety for patients and physicians. Unnecessary cables disappear, and with them potential hazards.

Improvements are also noticeable in ergonomic terms. Physicians and nursing staff can move more freely and are no longer “tied” to one place by numerous cable connections.

Modern Bluetooth radio transmission technology guarantees that safety is not compromised in any way. This technology offers all the features required to transmit medical technical data, whilst still adhering to the highest safety standards.

Interference resistance

steute enables users to operate different devices in parallel, without any interference. Besides numerous techniques for the automatic detection and correction of transmission errors, Bluetooth also provides additional safety by avoiding sources of interference using its frequency hopping technique.

The frequency band used by Bluetooth is divided up into 79 channels (Fig. 1), which can all be used for data transmission equally. During transmission these channels are permanently changing (Fig. 2), so that data batches with interference can be re-transmitted on another channel (with no interference).

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Configuration area

If two wireless devices use the same radio technology, they can communicate with each other and swap information. This feature, which at first sight seems extremely positive and which was the motivation behind the Bluetooth development, also involves a whole series of dangers.

Controls with wires are either firmly connected to a device or are at least coded by a particular plug or a particular configuration as belonging to one device series. If you take the cable away, you also lose this special device coding.

In order to ensure that transmitter and receiver definitely match, **steute** has included a configuration area in its RF/BT module where the origin and functions of each different module can be individually entered (Fig. 3). Besides classification data like manufacturer, device type and device class (e.g. foot control), the module's configuration data can also be stored. These are the number and function of digital and analogue I/Os, operational settings like alarm duration and warning delay, as well as the baud rate of the serial interface. With these configuration data the transmitter and the receiver can establish during the connection process whether they are compatible and whether safe operation is therefore possible.

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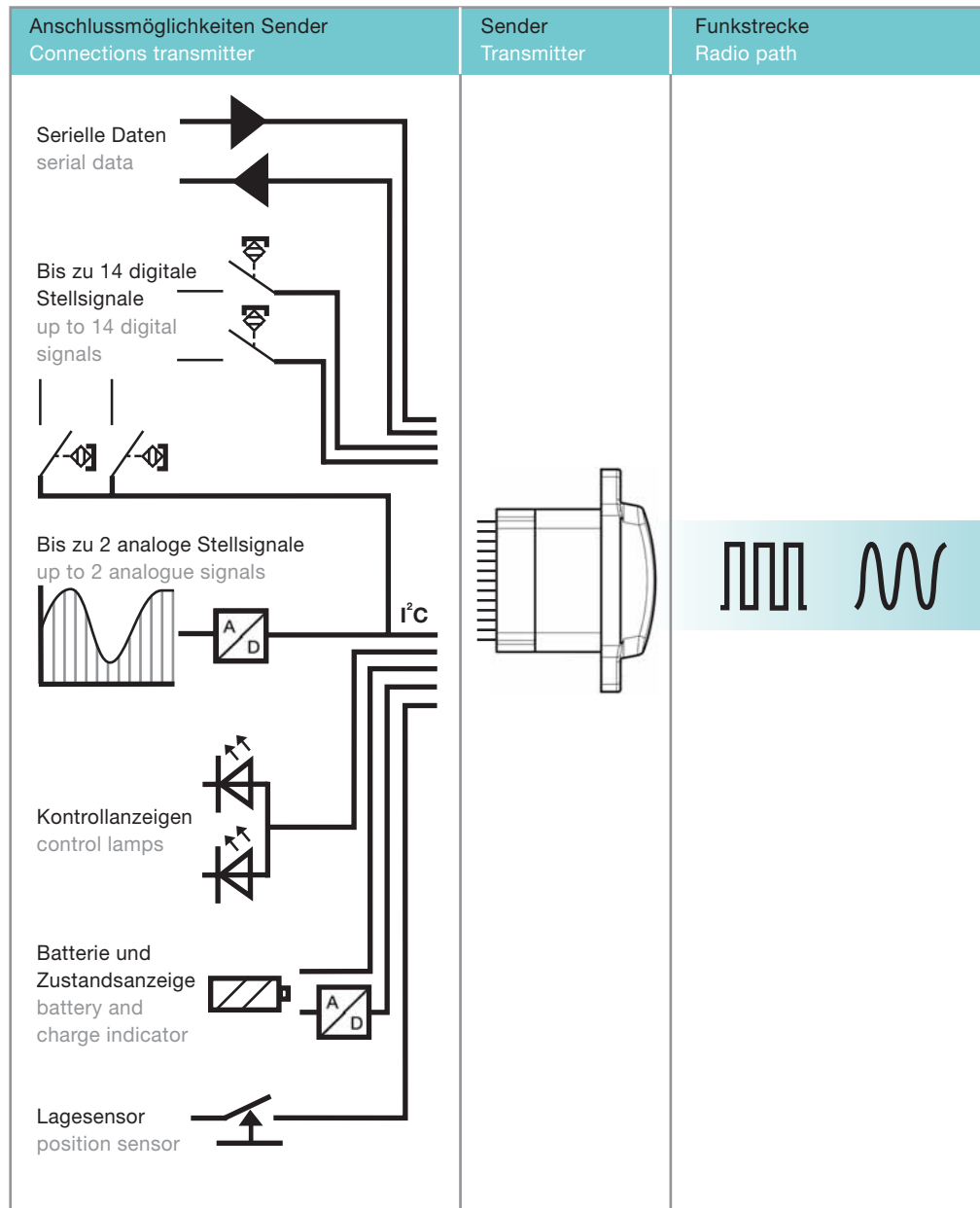
The Periphery

All input and output signals are handled at the edge of the board on a 26-pin connector. With a minimum of external configuration, up to 4 digital functions are possible.

Additional peripheral components can be connected via the I²C bus. In this way, up to 14 digital and 2 analogue signals can be transmitted. The maximum reaction time is 50 ms.

A serial interface with adjustable transmission speed (baud rate) is also integrated. Depending on the external configuration, asynchronous interfaces with cables (e.g. RS232) can be replaced by the RF/BT.

The module requires a stabilised voltage of 2.9-3.4V. Peak power consumption is approx. 150 mA.

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
Initialisation procedure

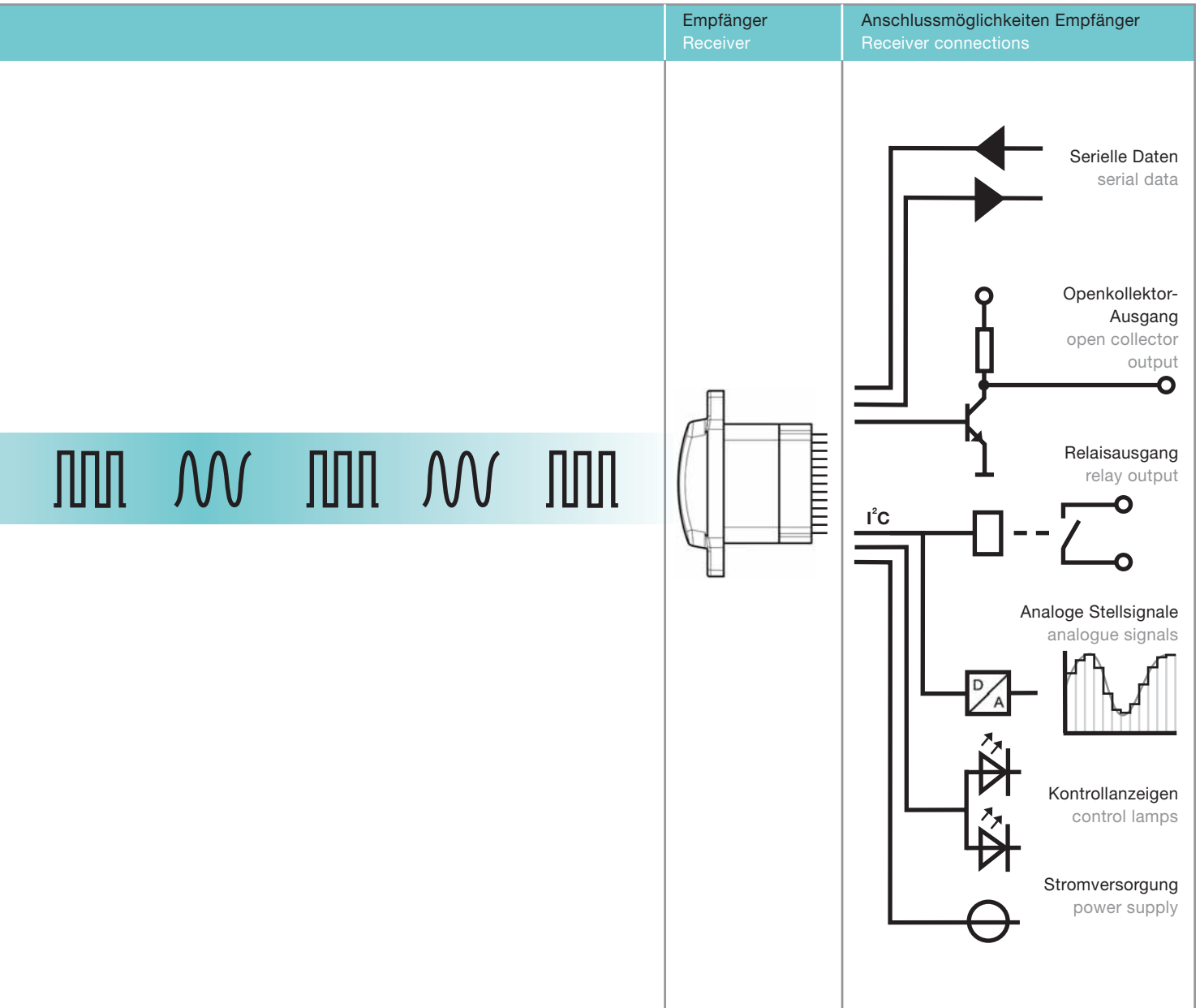In order to increase operational safety even further, **steute** has integrated a whole series of innovative functions. To enable Bluetooth controls to be “connected” ad hoc to any device as with a cable, **steute** employs a special initialisation procedure. Via an infrared path a second data channel is established, along which the transmitter and the receiver can exchange data. In addition to an individual and globally unique identification number, manufacturer and configuration details are also exchanged. In this way any incompatibility is detected before operation commences and can thus be effectively prevented.

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Sensors

Wireless foot controls have no cables to tie them down to one location. In order to rule out incorrect operation, in-house sensors permanently monitor the location of each control. If one is lifted off the floor, all functions are automatically blocked and a warning signal emitted. If the control is not replaced within a certain period (configurable between 1 and 60 seconds), radio transmission automatically ceases.

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Battery operation and charging procedure

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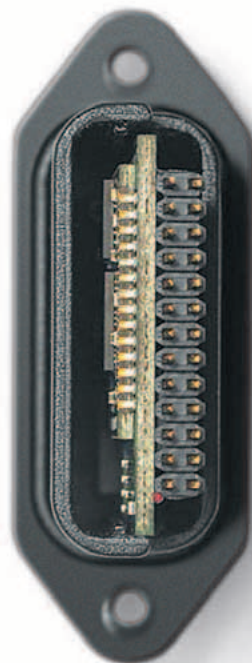
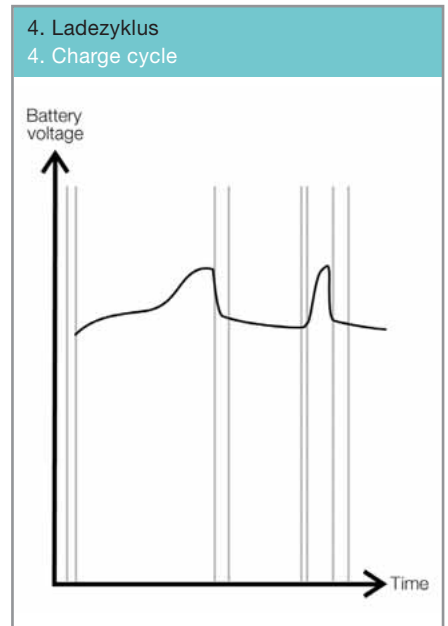
Battery operation

The wireless foot control is powered by a conventional rechargeable battery. Depending on the size of the control, different types of rechargeable batteries can be used, facilitating different operating times. In order to keep down electricity consumption, thus guaranteeing operation for as long as possible, energy-saving components were consistently employed throughout the development of the module. With a double-pedal foot control and three 1600 mAh cells, an operating time of approx. 10 hours can be achieved.

The way the battery is monitored can be adjusted to suit individual needs. From a simple digital display of the battery level or the monitoring of voltage, to complex measuring procedures with automatic actuation of a test load, different concepts suitable for all manner of applications have been incorporated.

An intelligent charging procedure additionally guarantees fast and careful charging, and thus longer battery life.

The charger used can charge all types of battery with extreme precision. Even partly empty batteries can be charged; the special charging procedure prevents a memory effect from occurring. Once the battery has been fully charged, the processor automatically switches to a careful trickle charge (Fig. 4).

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
RF/BT module	
Construction:	Bluetooth construction, qualified microprocessor system with an intelligent safety concept
Topology:	max. of 2 foot controls to one receiver
Transmission:	Bluetooth, globally recognised and authorised radio standard
Functions:	max. 14 digital and 2 analogue
Voltage:	2.9-3.4 V, 150 mA
Operating time:	approx. 20 to 40 h, depending on the foot control
Power:	1 mW
Range:	approx. 10 m
Response time:	< 50 ms
Analogue transmission:	8 bit resolution
Baud rate:	2400, 9600, 19200, 57.6k, 115k baud
Standards/Directives:	EN 60601-1, EN 60601-1-2, EN 60950, EN 301498-1, EN 301498-17, EN 61000-3-2, EN 61000-3-3, EN 300 328-2, CE(93/42/EEC), ISO 14971, UL 1950, FCC part 15, J60950, ARIB-STD-T66
Authorised for use in:	Europe, USA, Canada (other countries on request)
Safety functions:	<ul style="list-style-type: none"> - guaranteed safe parallel operation of several controls - special initialisation procedure facilitates easy removal of defect controls - permanent monitoring of radio path - monitoring of operational location through position sensors - automatic disconnection of foot control after configurable delay

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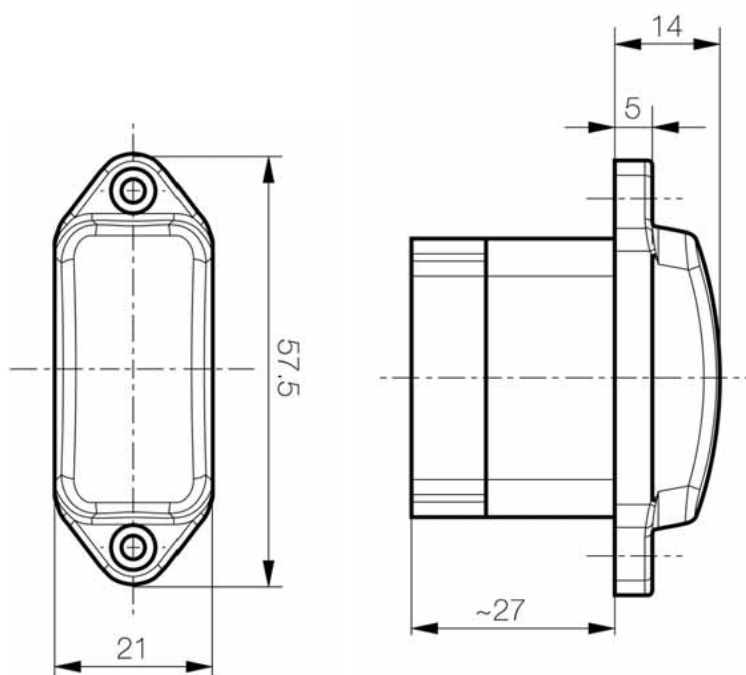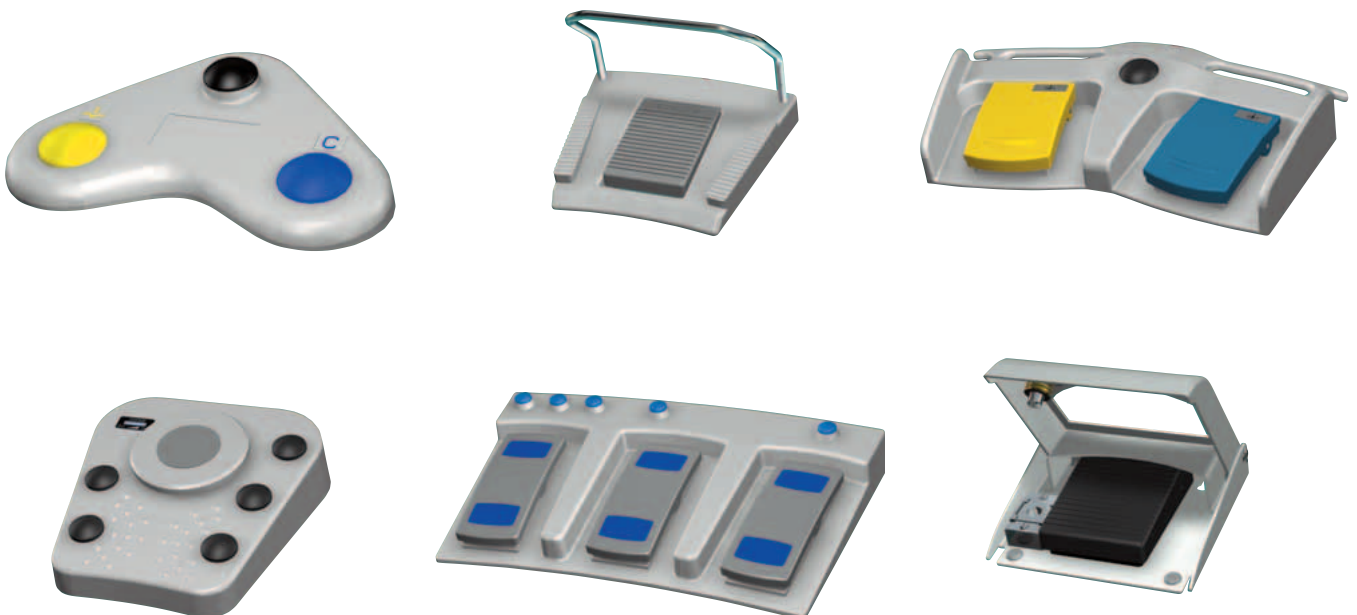
Ergonomics, safety and total reliability: these are the primary requirements of medical technical controls. Foot-operated controls are particularly important. When your hands are full and concentration is required, devices can best be operated using pedals. **steute** has been playing an active role in this complex market for 25 years and today sells not only a wide range of standard applications, but also numerous customised solutions.

Cooperation

Success in this corner of the market led to the founding of a new business field in 1996, dedicated exclusively to medical technical applications. Here we collaborate closely with our customers, but also with certification bodies and independent institutes such as the “Center for Ergonomy and Fitness for Use” and the “Center for Medical Technology” at Münster Polytechnic. Our RF/BT module was also developed in cooperation with these two institutes and sponsored by the NRW Ministry for Science and Research.

Intensive work with controls designed specifically for medical technical applications has led to numerous innovative solutions. One example is a foot control which can be operated from all sides, regardless of the position of the control relative to the operator. **steute** has also had extensive experience in the development and production of controls using wireless signal transmission, e.g. radio-controlled or infrared transmission.

Foot controls from **steute** Meditech are to be found all over the world – in hospitals, surgeries, rehabilitation centres and laboratories.

If you would like additional information or sales advice, please contact us directly or visit our website at www.steute.de.

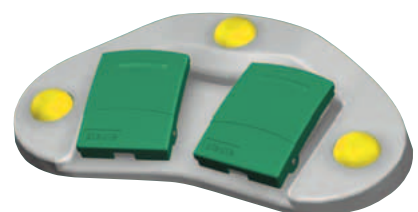
.steute

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University of
Applied Sciences

Ministry for
Science and
Research NRW

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Beside foot controls for medical equipment steute Schaltgeräte designs and produces switchgear for complex and critical applications in industry – for example pull-wire, door handle, foot switches and safety switchgear. Furthermore steute boasts a wide program of explosive protected switchgear that are applied where flammable atmospheres can occur – e. g. Ex position switches, Ex pull-wire switches, Ex safety switches and solenoid interlocks, as well as Ex magnetic sensors.

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Schaltgeräte GmbH & Co. KG